EN

**OSOM® Trichomonas Positive Control Kit**  
**Catalog number 182**

**INTENDED USE**

The OSOM® Trichomonas Positive Control Kit is intended for *in vitro* diagnostic use in quality control testing with the OSOM Trichomonas Rapid Test.

**SUMMARY**

The OSOM Trichomonas Positive Control Kit provides swabs coated with a dried mixture containing Trichomonas antigens. The level of Trichomonas antigen applied to the swab is sufficient to provide a positive result when used with the OSOM Trichomonas Rapid Test. Sterile rayon swabs supplied in the Trichomonas Rapid Test kit are free of Trichomonas antigens and can be used as negative controls. When used for quality control in the OSOM Trichomonas Rapid Test, these controls are useful as an aid in verifying test and operator performance.

**MATERIALS PROVIDED**

OSOM Trichomonas Positive Control Swabs (approximately 1 x 10<sup>5</sup> cell equivalents)- 10 polyester swabs coated with an antigen mixture derived from *Trichomonas vaginalis* organisms.

**MATERIALS REQUIRED BUT NOT PROVIDED**

•OSOM Trichomonas Rapid Test

•Clock or Timer

**WARNINGS AND PRECAUTIONS**

1. DO NOT use the Controls beyond the expiration date.

2. Potential biohazard. Handle as if potentially infectious. Handle this product according to established good laboratory practices using appropriate precautions.

3. Not for patient use.

**STORAGE**

•The OSOM Trichomonas Positive Control Kit must be stored at room temperature (15-30°C/59-86°F).

•Tightly close the canister after use to prevent deterioration of the positive control swabs.

**STABILITY**

Swabs stored in the closed canister at 15-30°C (59-86°F) are stable until the expiration date printed on the canister and box label.

**INSTRUCTIONS FOR USE**

Use the OSOM Trichomonas Positive Control Swabs as you would a patient sample in accordance with the procedure for the OSOM Trichomonas Rapid Test. Read the procedural instructions from the assay product insert prior to testing.

**EXPECTED RESULTS**

The Negative Control should yield a negative result as described in the Interpretation of Test Results section of the assay product insert.

The Positive Control should yield a positive result as described in the Interpretation of Test Results section of the assay product insert.

**DISPOSING OF MATERIALS**

Dispose of hazardous or biologically contaminated materials according to your institution’s practices. Discard all materials in a safe and acceptable manner that is in compliance with all country, state, and local requirements.

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OSOM® is a registered U.S trademark of SEKISUI Diagnostics, LLC. All other trademarks are the property of their respective owners.
SEKISUI Diagnostics, LLC 6659 Top Gun Street San Diego, CA 92121 USA