

# Metrix Training Checklist

Use as an aide in training operators on Metrix tests

It is recommended that testing be performed using controls or blank swabs.

Activities	Status (Completed, Missed, Not Applicable)
Trainer demonstrates how to run test and asks trainee to follow along, step by step.	
Trainer should refer to the test QRI (Quick Reference Instructions) and reader QSG (Quick Start Guide) as a resource.	
Trainer emphasizes that patient should blow their nose before sample is collected and operator should confirm that there is no excess mucus or blood on the swab prior to testing.  Note that for training purposes, testing should be performed using controls or a blank swab, not patient samples.	
Trainer reminds trainee to check that black lines are visible through openings on the cap (refer to Step 3 of QRI)	
Trainer reminds trainee to firmly insert collector into sensor. There should be no gap between the collector and white ring on sensor (refer to Step 5 of QRI)	
Trainer asks the trainee to perform a test on their own	
Trainer reviews test results using QRI (Step 7), reader sticker, or demonstration video ( <a href="https://sekisuidiagnostics.com/training/metrix-covid-flu-test/">https://sekisuidiagnostics.com/training/metrix-covid-flu-test/</a> )	
Trainer should explain how to handle test failures (errors, cancelled test and invalid results) using the “Troubleshooting” section of the QRI	
Verify that the results of tests performed during training are as expected (i.e.: blank swab should produce a negative result).  Note- If results are unexpected/invalid etc., it is recommended that the trainee perform another test on their own to ensure that the training was successful.	