



# POINT-OF-CARE



## OSOM® COVID-19 Test

### RELIABLY DETECT SARS-CoV-2 IN ONLY 15 MINUTES AT THE POINT-OF-CARE

#### Features and Benefits

- **Rapid** - Deliver test results to your patients in only 15 minutes, at the point-of-care
- **Reliable Testing** - Test patients of all ages within 4-days of symptom onset and stop delaying treatment
- **Easy-to-Use** - A simple, CLIA-waived procedure that doesn't require disposable pipettes for dispensing
- **Cost Efficient** - Unique QC Inside® feature that provides you with two additional test for external QC testing
- **Peace-of-Mind** - Manufactured in the USA so you can ease tariff concerns and avoid delayed product shipments
- **Exceptional Support** - Experienced support teams and online training modules to help streamline your testing implementation


#### Performance Characteristics

| Sample Type | Sensitivity | Specificity | Limit of Detection                       | Storage Temperature | Time to Result |
|-------------|-------------|-------------|--|---------------------|----------------|
| SARS-CoV-2  | 84.8%       | 99.7%       | $7.0 \times 10^2$ TCID <sub>50</sub> /mL | 2-30°C/35-86°F      | 15 minutes     |




SENSITIVITY

SPECIFICITY

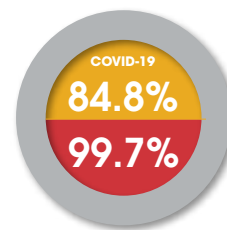

# OSOM® COVID-19 Test

Experience a Quick and Easy Test Procedure

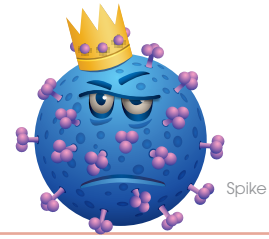

**1** INSERT THE COLLECTED SWAB INTO THE REAGENT TUBE.

**2** SWIRL AND PLUNGE UP AND DOWN WHILE SQUEEZING THE SIDES OF THE TUBE FOR 15 SECONDS.

**3** REMOVE THE SWAB WHILE SQUEEZING THE SIDES OF THE TUBE AND FIRMLY CLOSE THE DROPPER TIP ONTO THE TUBE.

**4** DISPENSE 2 DROPS OF SAMPLE FROM THE REAGENT TUBE INTO THE SAMPLE WELL.

**5** READ TEST AT 15 MINUTES.

## INTERPRETATION OF RESULTS

### Positive



### Negative



### Invalid



## Ordering Information

|                     | Configuration | Catalog Number | CPT Code |
|---------------------|---------------|----------------|----------|
| OSOM® COVID-19 TEST | 25 tests      | 1092           | 87811QW  |

To learn more about all of our OSOM Tests scan the code or go to [osomtests.com](https://osomtests.com)



**SEKISUI**  
DIAGNOSTICS

Because every result matters™

sekisuidiagnostics.com  
questions@sekisui-dx.com

