



## OSOM® RSV Test

### TEST FOR BOTH HIGH-RISK GROUPS, YOUNGER CHILDREN & ELDERLY ADULTS, FOR SEVERE RSV INFECTION

#### Features and Benefits

- **Rapid & Reliable** – Deliver high quality test results to your patients in only 15 minutes, at the point-of-care.
- **Flexible** – Properly diagnose and care for BOTH high-risk patient groups with intended use claims for younger children and elderly adults.
- **Easy-to-Use** – Increase your productivity with a simple, CLIA-waived procedure that comes with ready-to-use extraction buffer tubes.
- **Noninvasive** – Provide a more comfortable experience to your patients through an anterior nasal swab collection process.
- **Cost Efficient** – Unique QC Inside® feature that provides you with two extra tests per kit at no additional charge for external QC testing.
- **Peace-of-Mind** – Manufactured in the USA so you can receive cost-effective pricing, while reducing the risk of supply chain disruption.
- **Exceptional Support** – Streamline your testing implementation and ongoing staff trainings through online training modules and experienced support teams.

#### Performance Characteristics

	Sensitivity*	Specificity*	Sample Type	Limit of Detection**	Storage Temperature	Time to Result
RSV	83.8%	99.6%	Anterior nasal swab	$1.6 \times 10^3$ TCID <sub>50</sub> /mL	2-30°C/ 36-86°F	15 minutes

\*Combined prospective clinical RT-PCR comparator study.

\*\*LOD for RSV Subtype A. Refer to IFU for additional LODs.







SENSITIVITY  
SPECIFICITY



# OSOM® RSV Test

SEAMLESS SAMPLE PROCESSING WITH MINIMAL HANDS-ON TIME

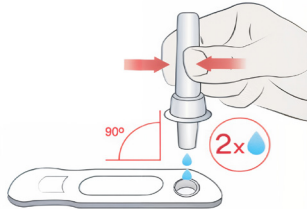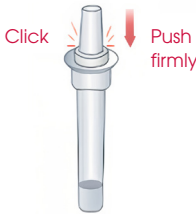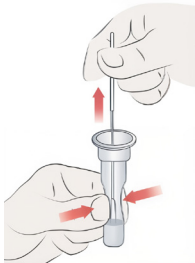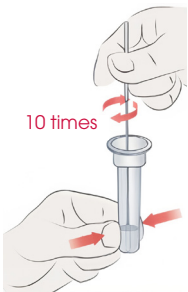

**1** STIR THE SWAB AT LEAST 10 TIMES INSIDE THE EXTRACTION BUFFER.

**2** REMOVE THE SWAB WHILE SQUEEZING THE SIDES OF THE TUBE.

**3** ATTACH THE PROVIDED DROPPER TIP TO CLOSE THE REAGENT.

**4** DISPENSE 2 DROPS OF SAMPLE INTO THE SAMPLE WELL OF THE TEST DEVICE.

**5** READ RESULTS AT 15 MINUTES.

## INTERPRETATION OF RESULTS

### Negative



### Positive



### Invalid



## Ordering Information

Description	Configuration	Catalog Number	CPT Code
OSOM RSV TEST	25 tests per kit	1091	87807QW

To learn more about all of our OSOM Tests scan the code or go to [osomtests.com](https://osomtests.com)



**THE AMERICAS**  
SEKISUI Diagnostics, LLC  
One Wall Street  
Burlington, MA 01803  
Phone: 800 332 1042  
Email: [questions@sekisui-dx.com](mailto:questions@sekisui-dx.com)

**SEKISUI**  
DIAGNOSTICS  
Because every result matters™  
[sekisuidiagnostics.com](https://sekisuidiagnostics.com)

